## Opti-sol XL 1500 Cleaner

Cat. No.	Product Name	Packaging (Content)
REG00055	Opti-sol XL 1500	1 × 400 mL, Instruction for use, RFID tag

### INTENDED USE

Opti-sol XL 1500 is the cleaning solution that reduces the surface tension of the deposited sediment particles and prevents the microbial growth in cuvettes and hydraulic system of automatic urine analyzer LAURA XL intended for *in vitro* diagnostics. For professional use at clinical laboratories only.

### PRINCIPLE

The performance of analyser LAURA XL depends on stabilization and cleanest purity of optical and flow parts. Opti-sol XL 1500 solution contains additives which stabilize optical and fluidic parts and provides clean environment in the system.

### DESCRIPTION AND COMPOSITION

Wetting agents, emulsifiers, complexing agents, remaining substances 70 %  
Tripotassium orthophosphate 30 %

### WORKING SOLUTION PREPARATION

The content of the bottle of Opti-sol XL 1500 dilute in 10 litres of purified water (400 mL of Opti-sol XL 1500 + 9600 mL of water) into provided tank and attach to LAURA XL analyser.

### MATERIAL REQUIRED BUT IS NOT PROVIDED WITH THE DEVICE

LAURA XL automatic urine analyser, Cat. No. INS00065

### STABILITY AND STORAGE

The cleaning solution is stable till the expiry date mentioned on the label. The cleaning solution should be stored in tightly closed original packaging in a dry and dark place at a temperature 2–25 °C.

### WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and / or the patient is established.

### Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: 92FW-WWNP-YJ5M-VCKU

Warning

**Contains:** Tripotassium orthophosphate, potassium hydroxide

#### Hazard statement:

H290 May be corrosive to metals.  
H315 Causes skin irritation.  
H318 Causes serious eye damage.  
H335 May cause respiratory irritation.  
H412 Harmful to aquatic life with long lasting effects.

#### Precautionary statement:

P261 Avoid breathing mist/vapours/spray.  
P264 Wash hands and exposed parts of the body thoroughly after handling.  
P280 Wear protective gloves/protective clothing/eye protection.  
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
P312 Call a POISON CENTER/doctor if you feel unwell.

**Contains:** 15 – <30 % phosphates, <5 % anionic surfactants, <5 % non-ionic surfactants

#### Working solution:

Working solution is not classified as dangerous.

### WASTE MANAGEMENT

Please refer to local legal requirements.

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