ELite H580 Lyse3

Haematology Analyser Lyse

Cat. No.	Product Name	Packaging (Content)
HEM00022	ELite H580 Lyse3	1 L, Instruction for use, RFID card

INTENDED USE

ELite H580 Lyse3 is an *in vitro* diagnostic reagent used for quantitative 5-part differentiation of white blood cells on automated haematology analysers ELite 580. It is designed to report on the haematological status of patient samples, supporting the monitoring and diagnosis of pathological conditions. The ELite H580 Lyse3 should be used by trained professionals in the haematology laboratory.

PRINCIPLE

The reagent is used with the ELite H580 Lyse2 to dissolve the RBC to change the morphology of WBC, so as to test WBC classification result.

DESCRIPTION AND COMPOSITION

Surfactant; Buffering Agent; Antifungal and antibacterial agents.

PREPARATION

This reagent is ready for use and can be applied straight from the container; no special reagent preparation is necessary. Avoid reagent contaminations. Before using, leave the reagent at room temperature for at least 24 hours. For reagent connection to the analyser, refer to the ELite 580 User manual.

MATERIAL REQUIRED BUT IS NOT PROVIDED WITH THE DEVICE

Haematology analyser ELite 580, Cat. No. INS00071

Reagents: ELite H580 Dil, Cat. No. HEM00019

ELite H580 Lyse1, Cat. No. HEM00020

ELite H580 Lyse2, Cat. No. HEM00021

SPECIFICATION

The product is a clear liquid without particles, precipitate or floccule. Background result: WBC $\leq 0.2 \times 10^9/L$, RBC $\leq 0.02 \times 10^{12}/L$, HGB $\leq 1 g/L$, PLT $\leq 10 \times 10^9/L$.

STABILITY AND STORAGE

The product is stable till the expiry date mentioned on the label when stored at (+2 to +30) °C. Discard opened container after 60 days.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and / or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: 3CDF-W3E9-NN47-2P1C

Warning

Hazard statement:

H315 Causes skin irritation.

H319 Causes serious eye irritation.

Precautionary statement:

P280 Wear eye protection.

P302 + P352 IF ON SKIN: Wash with plenty of water and soap.

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lens, if present and easy to do. Continue rinsing.

P337 + P313 If eye irritation persists: Get medical advice/attention.

Supplemental information:

EUH208 Contains reaction mass of 5-chloro-2-methyl-4-isothiazolin-3-one and 2-methyl-2H-isothiazol-3-one. May produce an allergic reaction.

WASTE MANAGEMENT

Please refer to local legal requirements.

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