APOLIPOPROTEIN CALIBRATOR SET

Lot: xxxxxxxx

Cat. No.	Pack name	Packaging (Content)
XSYS0105	APO CAL SET	5 × 0.5 ml

INTENDED USE

Set of calibrators for immunoturbidimetric determination of apolipoproteins A1 and B in human serum and plasma (5 × 0.5 ml).

COMPOSITION

APO CAL SET is a liquid calibration material made of human serum mixture. It is used for calibration during determination of apolipoproteins A1 and B in human serum and plasma.

STORAGE AND STABILITY

The preparation is ready for use. It is treated as standard according to the instruction included to the diagnostic set. If APO CAL SET is stored correctly at 2–8 °C, it is stable till the expiration date stated on the cover. After use the vials must be screwed tightly and stored at 2–8 °C. Avoid contamination, pipette and decantation of standard back to the vial must be excluded.

PRECAUTIONS AND WARNING

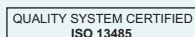
The set is specified only for use *in vitro*. To be handled by entitled and professionally educated person. Serum was tested for presence of HBsAg and antibodies against HIV and HCV with negative results. In spite of the fact, the presence of some pathogenic materials cannot be excluded. For this reason the content of the set has to be treated as potentially infectious material. Reagents of the kit are not classified like dangerous.

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