## Reagent for PHOSPHATASEtest

Cat. No.	Product Name	Packaging (content)
MLT00018	Reagent for PHOSPHATASEtest	Reagent for PHS: 1 × 18 mL, Instruction for use

### INTENDED USE

Reagent for PHOSPHATASE test is an *in vitro* diagnostic reagent used as supplementary colour forming reagent for some of MIKROLATEST® kits with pure bacterial cultures obtained from various clinical materials in the general population. The kits are dedicated for visual evaluation or for determination on automatic system ErbaExpert. The kit is intended as assistance in establishing a diagnosis or pathological state which can be caused by the tested pathogen. For professional use in clinical laboratory only.

### PRINCIPLE

Reagent for PHOSPHATASE test is colour forming reagent for test for the demonstration of phosphatase and serves for colour expression of this test.

### DESCRIPTION AND COMPOSITION

NaOH  
Purified water

### WORKING SOLUTION PREPARATION

Ready to use. Avoid to contamination.

### STABILITY AND STORAGE

The unopened Reagent for the PHOSPHATASE test is stable till the expiry date stated on the bottle and kit label when stored at (+2 to +25) °C. Store away from direct sunlight.

### ASSAY PROCEDURE

The Reagent for PHOSPHATASE test should be used in accordance with the instructions for use of the specific kit.

### WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### Hazards identification in accordance with Regulation (EC) No 1272/2008

#### Warning

Contains: Sodium hydroxide, (Index: 011-002-00-6; CAS: 1310-73-2)

#### Hazard statement:

H315 Causes skin irritation.

H319 Causes serious eye irritation.

#### Precautionary statement:

P280 Wear protective gloves/protective clothing/eye protection.

P302+P352 IF ON SKIN: Wash with plenty of water.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

### WASTE MANAGEMENT

Please refer to local legal requirements.

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