Reagent for INDOLtest

Cat. No.	Product Name	Packaging (Content)
MLT00020	Reagent for INDOLtest	Reagent for IND: 1 × 18 mL, Instruction for use

INTENDED USE

Reagent for INDOL test is an *in vitro* diagnostic reagent used as supplementary colour forming reagent for some of MIKROLATEST® kits with pure bacterial cultures obtained from various clinical materials in the general population. The kits are dedicated for visual evaluation or for determination on automatic system ErbaExpert. The kit is intended as assistance in establishing a diagnosis or pathological state which can be caused by the tested pathogen. For professional use in clinical laboratory only.

PRINCIPLE

Reagent for INDOL test is colour forming reagent for test for indole production and serves for colour expression of this test.

DESCRIPTION AND COMPOSITION

4 – Dimethylaminobenzaldehyde
Ethanol anhydrous, denatured 5 % methanol
HCL 35%

WORKING SOLUTION PREPARATION

Ready to use. Avoid to contamination.

STABILITY AND STORAGE

The unopened Reagent for the INDOL test is stable till the expiry date stated on the bottle and kit label when stored at (+2 to +8) °C. Store away from direct sunlight. We recommend to use Reagent for the INDOL test within 9 months after opening.

ASSAY PROCEDURE

The Reagent for INDOL test should be used in accordance with the instructions for use of the specific kit.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: DSX7-EJCW-KE7D-EMAP

Danger

Contains: hydrochloric acid 35%, 4-dimethylaminobenzaldehyde, methanol

Hazard statement:

H225 Highly flammable liquid and vapour.
H315 Causes skin irritation.
H317 May cause an allergic skin reaction.
H319 Causes serious eye irritation.
H335 May cause respiratory irritation.
H371 May cause damage to organs.
H410 Very toxic to aquatic life with long lasting effects.

Precautionary statement:

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
P273 Avoid release to the environment.
P280 Wear protective gloves/protective clothing/eye protection.
P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.
P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P308 + P311 IF exposed or concerned: Call a doctor.

WASTE MANAGEMENT

Please refer to local legal requirements.

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