NITROCEFİN

Kat. č.: MLT00031

Pro mikrobiologii

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Principle:

Nitrocefín is a chromogenic cephalosporin that has been found to be effective in detection of all known β-lactamase enzymes. β-lactamase enzymes hydrolyse the amide bond in the β-lactam ring of Nitrocefín resulting in a distinctive colour change from yellow to red.

A package contains:

- 50 discs
- Working leaflet

Storage, expiration:

NITROCEFİN discs should be stored in the freezer at below -10°C on receipt in the containers provided until the expiration date indicated on the package. Allow to equilibrate to room temperature before opening. Return to freezer storage immediately after use.

Caution:

- For professional use only.
- After use all discs must be autoclaved or incinerated.

Required material:

- empty Petri dish or microscope slide
- common laboratory equipment (loops, sterile forceps, burner)

Recommended procedure:

1. Place the required number of NITROCEFİN discs into an empty Petri dish or onto a microscope slide.
2. Disc may be moistened with one drop of deionised water. Do not over-moisten.
3. Using a sterile loop or applicator stick remove several well-isolated colonies and smear onto the surface of a disc.
4. Observe disc for colour change.
5. Alternatively: moisten the disc with one drop of deionised water, then holding the disc in forceps wipe it across a colony on an agar plate.

Interpretation:

- A positive reaction will show as a yellow to red colour change on the area where the culture was applied.
- Note: the colour change does not normally develop over the whole of the disc.
- A negative result will show no colour change on the disc.
- For most bacterial strains a positive result will develop within 5 minutes. However, positive reactions for some staphylococci and anaerobic species may take up to 60 minutes to develop.
- Some strains of *Staphylococcus aureus* produce small amounts of the β-lactamase enzyme unless enzyme production is induced by growth in a medium containing sub-inhibitory concentration of β-lactam antibiotic.
- A positive result should be interpreted as resistance to penicillin or cephalosporin activity. Susceptibility should be confirmed by standard growth-dependent susceptibility testing methods.
- Negative results imply but do not guarantee susceptibility.

Control strains:

Strain	Control
Staphylococcus aureus CCM 4223 (ATCC 29213)	positive
Staphylococcus aureus CCM 3953 (ATCC 25923)	negative

These strains are supplied by the CCM – Czech Collection of Microorganisms, Masaryk University, Faculty of Science, Kamenice 5, building A25, 625 00 Brno, CZ, tel. 549 491 430, fax 549 498 289, <http://www.sci.muni.cz/ccm>, e-mail: ccm@sci.muni.cz

Strains are supplied in freeze-dried form or on gelatin discs.

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