OFtest

Kat. č.: MLT00032

Pro mikrobiologii

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Cat. No.: MLT00032

For microbiology

OFtest is intended for rapid differentiation of fermentative and oxidative glucose metabolism. OFtest is placed in the wells of single strips of breakable microwell plate. One plate enables up to 96 examinations.

The OFtest contains:

- Breakable microwell plate with desiccant, 3 pcs.
- Instructions leaflet
- Bags for incubation, 3 pcs.
- A storage bag for an opened plate

Storage, expiration:

The OFtest kit should be stored at temperature (+2 to +8) °C. Expiration is indicated on the each package.

RECOMMENDED PROCEDURE

Precautions:

Kit is intended for professional use only. The kit should be used only by individuals being skilled in principles of work with infectious material and safety disposals according to local obligatory directions of laboratory.

Material required for work with the OFtest kit:

- Petri dishes with Columbia blood agar, Trypton – soya agar, nutrient agar or other suitable non-selective cultivation medium
- Paraffine oil sterilized, Cat. No. MLT00042 - 180 determinations
- Tubes (100x15) mm with sterile non-buffered saline solution
- Instrument Densi-La-Meter II, Cat. No. INS00062
- Vortex V1, Cat. No. 50001715
- Stepper pipette Mikro-La-Stepper, Cat. No. 50001707
- Thermostat 37° C
- Common equipment of microbiological laboratory (loop, burner, marking pen etc.)

Recommended procedure:

- Prepare bacterial suspension from a pure 24 hours culture in non – buffered saline solution. The turbidity should be at least 4 McF. Homogenize the suspension well, preferably by Vortex.
- Open the aluminium bag and take out the plate. Break out required number of wells from the plate.
- Put the rest of the plate to the storage bag with desiccant and store it in a refrigerator for the further use. Protect it from humidity. It is recommended to use the rest of the wells till 4 weeks after opening.
- Place the wells into empty frame when working with the kit first time and an empty plate is not available, store unused wells in the storage bag freely and use the frame for incubation.
- Inoculate 100 µl of prepared suspension into the wells.
- After inoculation overlay the wells with 3 – 4 drops of paraffine oil.
- Use a PE bag for incubation. Fold the opened side of the PE bag down to avoid desiccation of inoculum.
- Incubate in thermostat at + 37 °C for 2 – 4 hours.

Note:

Any uneven distribution of substrate in the well does not affect the functionality of the test.

Interpretation of the reaction:

Reaction	Colour expression
positive	yellow
negative	green

The kit characteristics:

The kit was tested on 50 strains. 93.3 % was identical to conventional test done in tube 6.7 % gave different result

The most frequent cause of identification failure:

- Violation of some step of recommended procedure

Disposal of used material:

After use put the panel in container intended for infectious material or autoclave or incinerate.

Quality control of OFtest:

For internal quality control it is recommended following control strains: CCM 1903 *E. cloacae* (positive control) and CCM 3919 *Burkholderia cepacia* (negative control). Use only fresh isolates of the CCM strains each time when you check the correct functioning of the kit.

These strains are supplied in by the Czech Collection of Microorganisms, Masaryk University, Kamenice 5, 602 00 Brno, Czech Republic. Phone: (+420) 549 491 430, Fax: (+420) 543 247 339, e-mail: ccm@sci.muni.cz. The strains are supplied lyophilised or on gelatin discs.

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