PYRAtest

Kat. č.: MLT00040

Pro mikrobiologii

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Cat. No.: MLT00040

For microbiology

The PYRAtest is intended for a rapid detection of the pyrrolidonyle arylamidase (PYRase, test PYR) for diagnostic purposes in microbiology. The test serves especi-
ally for approving the pertinence to the family of *Enterococcus*, besides, the positive test is typical for *Streptococcus pyogenes* or it can be used in the capacity
of a differentiation test for representatives of the strain *Enterobacteriaceae*.
The test can be applied individually or as an complementary test to identification kits MIKROLATEST®. One package of the PYRAtest enabling 50 determinations.

Principle:
The bacterial pyrrolidonyle arylamidase hydrolyzes β-naphthylamine of the pyroglutamic acid which is included in the zone of the detection strip. The hydrolysis is
detected by the reaction with p-dimethylaminocinamaldehyde contained in the reagent solution for the PYR test under the appearance of a red colouration.

- One package of PYRAtest contains:**
- 50 detection strips for 50 determinations
 - working instructions
 - Storage:**
PYRAtest should be stored at the temperature of (+2 to +8) °C. Store away from direct sunlight. The expiry date is indicated on each package.
 - Required material:**
• Reagent for the test PYR (Cat. No. MLT00023)
 - Usual laboratory equipment (inoculation loops, burner)

Caution:

- For professional use only

- Working procedure:**
- Use a 24 hours old culture from a suitable cultivation medium (blood agar for enterococci, resp. *Streptococcus pyogenes*)
 - Moisten the strip zone with approx. 20 microlitres of distilled water
 - Using the inoculation loop, rub in several colonies of the tested culture into the strip zone, when using a pure culture, the bacterial growth can be wiped off from the agar surface directly by the strip zone.
 - Place the strip on a suitable support plate and incubate for approx. 10 minutes at the laboratory temperature.
 - Drip approximately 1 drop of the reagent solution for the PYR test onto the strip zone using the dropper (included in bottle).
 - Evaluate the colour reaction after 1–2 minutes directly on the strip zone, according to the table Evaluation of the PYR test.
 - In the case of strains giving a slightly positive, rather uneasily interpretable reaction, a variant of the test with the prolonged incubation can be used as follows:
 - Prepare a suspension of a density corresponding to the 3rd stage of the McFarland's turbidity scale from the tested culture in a test tube (100x15) mm with 0.5-1 ml of the sterile physiological solution.
 - Insert a PYRAtest strip into the suspension prepared in the described way (the strip zone has to be submerged in the suspension) and incubate in a thermostat for the period of 4 hours at the temperature of 37 °C.
 - After the incubation period has elapsed, drop approx. 6 drops of the reagent for the PYR test into the suspension (using the dropper included in bottle) and interpret the colour reaction of the suspension according to the table Evaluation of the PYR test.

Waste disposal:
Used strip should be treated as potentially infectious and should be liquidated in accordance with any other local and national regulations relating to the safe
handling of such materials. Put packaging waste to recycling or in municipal waste.

Evaluation of the PYR test

Reaction	Colour expression of reaction
positive	red, red-to-orange, orange
negative	yellow

Note:
For to enable the interpretation of the negative reaction, a negative control can be used, i.e.it is possible to use a strip without the bacterial culture layer parallelly
with the tested culture, resp. in the case of using the variant of the test noted for the prolonged incubation, the strip can be inserted into a physiolo-
gical solution without any bacterial suspension.

- Caution:**
- Observe the principles for working with infectious material.
 - Avoid any contact of the skin with the strip zone, after having finished the work, wash your hands thoroughly with water and soap.

Quality control of tests:

For to check and control the quality of the strips as well as the interpretation of the positive and negative reaction, it is possible to apply a test with
control strains:

Control Strains

Strains	CCM	ATCC	Reakce
<i>Enterococcus faecalis</i>	4224	29212	positive
<i>Streptococcus agalactiae</i>	6187		negative

These strains are distributed by
Czech Collection of Microorganisms, Masaryk University,
Faculty of Science, Kamenice 5, building A25, 625 00 Brno, CZ
Tel: +420 549 491 430, Fax: +420 549 498 289
http://www.sci.muni.cz/ccm, e-mail: ccm@sci.muni.cz
The strains are supplied lyophilised or on gelatin discs.

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