IVD Suspenzní médium MIC 200 (CZ)

Kat. č.: MLT00074

Pro mikrobiologii

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Cat. No.: MLT00074

For microbiology

Suspension Medium MIC 200 is supplementary preparation for MIC kits (MIC G-I, G-II, URINE, NEFERM, STAPHY), which are designed for antibiotic susceptibility testing.

Suspension Medium MIC 200 is supplied in glass bottles; the content of one bottle is 200 ml. The preparation of the suspension should be carried out according to the procedure recommended for a particular type of MIC test. The preparation of suspension is also included in the instructions for ErbaFill.

Content:

2 bottles with suspension medium MIC (200ml)
Instructions for use

Storage, expiration:

Suspension medium for MIC should be stored at the temperature of (2–25)°C. Store away from direct sunlight. Date of expiry is indicated on each package.

Quality control:

Aseptically transfer 5 ml of Suspension medium MIC 200 into tube. Prepare suspension of control strain with density of McF 0.5. Transfer 60 µl of suspension into suspension medium MIC in tube. Incubate at 35±2 °C for 16–20 hours.

Escherichia coli CCM 3954 (ATCC 25922) – turbid growth

Pseudomonas aeruginosa CCM 3955 (ATCC 27853) – turbid growth

Staphylococcus aureus CCM 4223 (ATCC 29213) – turbid growth

Warning and precautions:

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Hazards identification in accordance with Regulation (EC) No 1272/2008

Reagent is not classified as dangerous.

Waste disposal:

The residue of the suspension medium should be disposed in accordance with local and national regulations related to the safe handling of such materials. Recycle packaging paper waste.

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