BACITRACIN 10 UI

Kat. č.: MLT00083

Pro mikrobiologii

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Cat. No.: MLT00083

For microbiology

Diagnostic discs BACITRACIN 10 UI are intended for selective isolation of *Haemophilus* spp. in primary culture.

Principle:

Test is based on the resistance of *Haemophilus* spp. to high concentration of bacitracin (10 UI) in comparison with accompanying flora, and on the satellite growth of *Haemophilus* spp. in the zone of diffusion of exogenous growth factors from culture of staphylococci.

A package of the BACITRACIN 10 UI contains:

- 100 diagnostic discs for 100 determinations
- working leaflet

Storage, expiration:

Discs BACITRACIN 10 UI should be stored in dark in a refrigerator at (+2 to +8)°C until the expire date indicated on the package.

Required material:

- Blood agar plates
- common laboratory equipment (loops, sterile forceps, burner)
- 24 h culture of *Staphylococcus aureus*

Recommended procedure:

- Streak blood agar plate heavily with nasopharyngeal swab or with other clinical material.
- Apply a cross-streak of *Staphylococcus aureus* to the inoculated area
- Aseptically place disc BACITRACIN 10 UI on the staphylococcal line
- Incubate in an atmosphere containing (5–10)% CO₂ at 37 °C for (18–24) h.

Note:

Suitable combination of diagnostic discs (V+K DISK, BACITRACIN 10 UI, OPTOCHIN) may provide the determination of majority of important pathogens from upper respiratory tract in a single sample.

Evaluation:

Bacterial colonies achieved after incubation in the zone of bacitracin along the length of staphylococcal growth are suspected *Haemophilus* spp. Further determining of isolated strain can be provided by means of discs X, V a X+V- FAKTOR.

Caution:

- For professional use only.
- After use all discs must be autoclaved or incinerated.

Standard control strains:

Quality control of diagnostic discs BACITRACIN 10 UI may be provided with standard control strains:

Strain	Satellite effect	Inhibition zone
<i>Haemophilus influenzae</i> CCM 4457	+	growth
<i>Streptococcus pneumoniae</i> CCM 4501	–	growth inhibition >17 mm

These strains are distributed by:

CCM – Czech Collection of Microorganisms, Masaryk University, Faculty of Science, Kamenice 5, building A25, 625 00 Brno, CZ
Tel. +420 549 491 430, Fax +420 549 498 289, <http://www.sci.muni.cz/ccm>, e-mail: ccm@sci.muni.cz
Strains are supplied in freeze-dried form or on gelatin discs.

Date of revision: 18. 11. 2020

USED SYMBOLS

REF	Catalogue number	IVD	In vitro diagnostics		Manufacturer		See instruction for use
LOT	Lot number		Storage temperature		Expiry date		

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