LYONORM CALIBRATOR

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Characteristic

LYONORM CALIBRATOR is a lyophilized calibration serum based on human matrix. The concentrations and activities have been set to cover normal or slightly increased levels. The Calibrator is suitable for both manual methods and automatic analyzers.

Calibration values

Calibration values are specified in Certificate the number of which has to correspond to the lot number given on a label of each vial. LYONORM CALIBRATOR is secondary calibration material. Calibration values were obtained with use of analytic method mentioned in atest in standard conditions. Calibration values were specified as the median of the values obtained in seven independent laboratories in three independent series.

Reference

Morgenthaler, J. J.: Transfusion 27, 369 (1987)

Reagents

Calibrator (10 vials)

Preparation

Open a vial carefully, avoiding any loss of the lyophilized matter and add exactly 3.00 ml of diluent from redistilled or deionized water. Close vial carefully and dissolve the contents completely within 30 minutes at (+15 to +25) °C by swirling, avoiding the formation of foam. Do not shake. The Calibrator must be handled as carefully as a patient specimen.

With such diluted calibrator, it is possible to determine all serum analytes with exception of bicarbonate, sodium, potassium and chlorides.

Stability and storage

When stored at a dark place, the LYONORM CALIBRATOR is stable in the lyophilized form at temperature of (+2 to +8) °C till the date of expiration stated on the vial.

The expiry date for the lyophilizate is given on each vial.

The components of the reconstituted Calibrator (with exception of bilirubin and phosphatases) are stable for at least:

- 8 hours at (+15 to +25) °C
- 2 days at (+2 to +8) °C
- 4 weeks at (-25 to -15) °C when frozen once

Acid phosphatases (total and prostatic):

- 4 hours at (+15 to +25) °C
- 1 day at (+2 to +8) °C
- 2 weeks at (-25 to -15) °C when frozen once

Total bilirubin (when stored in the dark):

- 6 hours at (+15 to +25) °C
- 1 day at (+2 to +8) °C
- 2 weeks at (-25 to -15) °C when frozen once

Direct bilirubin (when stored in the dark):

- 3 hours at (+15 to +25) °C
- 8 hours at (+2 to +8) °C
- 2 weeks at (-25 to -15) °C when frozen once

The timing of stability is at 25 °C, when unfrozen!

Health protection

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Reagents of the set contain biological material, which is potentially infectious. The Calibrator has been prepared exclusively from the blood of donors tested to be free from antibodies of HIV (human immunodeficiency virus, also known as HTLV III/LAV), HCV (hepatitis C virus) and HB_sAg.

However, testing for HIV antibodies in serum or plasma as well as in the products thereof is less reliable. As the risk of the infection cannot be excluded, it is necessary to handle with the Calibrator just as carefully as with a patient specimen.

While working it is especially significant to observe sanitary rules, not to eat, drink, or smoke and to use self preventatives.

Waste disposal

The set should be treated as potentially infectious and the contingent rest of the reagent should be liquidated in accordance with any other local and national regulations relating to the safe handling of such materials.

Put packaging paper waste to recycling or in municipal waste.

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