CHEM WASH

Cat. No.	Product Name	Packaging (Content)
BLT00079	CHEM WASH	4 × 50 mL, Instruction for use

INTENDED USE

CHEM WASH is washing solution containing a surfactant that solubilises proteins and cleans the flowcell and cuvettes of Erba Semi-automated Clinical Chemistry Analysers intended for *in vitro* diagnostics. For professional use in clinical laboratory only.

PRINCIPLE

The performance of any Erba semi-auto analyzers depends on the cleanliness of the micro cuvette or flow through cell. Regular use of serum, plasma or urine results in protein deposition in the flow cell over a period of time. This interferes with the assay causing erroneous results. CHEM WASH contains a surfactant that solubilises proteins and cleans the flowcell without interfering with assay.

REAGENT DESCRIPTION AND COMPOSITION

Sodium hydroxide	<0.01 %
Polyethylene glycol dodecyl ether with >5–20 mol ethylene oxide	<0.01 %
Emulsifier K30	<0.01 %

REAGENT PREPARATION

Ready to use.

MATERIAL REQUIRED BUT IS NOT PROVIDED WITH THE DEVICE

CHEM 7 - semi-automated analyzer

STABILITY AND STORAGE

The reagent is stable till the expiry date mentioned on the label when stored at room temperature 15–25°C.

ASSAY PROCEDURE

Aspirate the CHEM WASH into the flow cell at the end of the day or whenever the instrument is not to be used for a long time. Flush the cell thoroughly with distilled water before reusing the instrument.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and / or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

The reagent is not classified as dangerous.

WASTE MANAGEMENT

Please refer to local legal requirements.

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