XL ISE Cleaning

Cat. No.	Pack Name	Packaging (Content)
XSYS0062	XL ISE CLEAN	1 × 52 mL, Instruction for use

INTENDED USE

XL ISE CLEANING is an aqueous enzyme mixture, used for cleaning the fluid lines of the ISE module of automatic analysers ERBA XL intended for *in vitro* diagnostics. For professional use in clinical laboratory only.

PRINCIPLE

The performance of any Erba XL analyzer ISE module depends on the cleanliness of the fluidic system. The use of serum or plasma or urine samples results in protein deposition. Over a period of time this may interfere with the assay resulting in erroneous results. XL ISE CLEAN contains a surfactant which solubilises proteins and cleans the fluidic system of ISE module.

REAGENT DESCRIPTION AND COMPOSITION

Protease (Bacterial)	0.06 µg/m ³
Propylene glykol	>3 %

REAGENT PREPARATION

Ready to use.

MATERIAL REQUIRED BUT NOT PROVIDED WITH THE DEVICE

Erba XL – analyzer with ISE

STABILITY AND STORAGE

The reagent is stable till the expiry date mentioned on the label when stored at room temperature 15–25 °C. Before use, keep the reagent at room temperature for at least 2 hours.

ASSAY PROCEDURE

Dispense 500 µL of cleaning solution into the XL sample cup and place on the ISE2 position of the sample tray. The cleaning solution should be run once a day to prevent protein build up or at 8 hrs intervals if the ISE module performs more than 50 samples per day.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and / or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: CMHN-DSQS-QN5T-8FSX

Danger

Contains: subtilisin

Hazard statement:

H315 Causes skin irritation.

H318 Causes serious eye damage.

H334 May cause allergy or asthma symptoms or breathing difficulties if inhaled.

Precautionary statement:

P261 Avoid breathing mist/vapours/spray.

P280 Wear protective gloves/protective clothing/eye protection.

P302 + P352 IF ON SKIN: Wash with plenty of water and soap.

P304 + P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing.

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P342 + P311 If experiencing respiratory symptoms: Call a POISON CENTER/doctor.

WASTE MANAGEMENT

Please refer to local legal requirements.

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