XL CLEANER

Cat. No.	Pack Name	Packaging (Content)
XSYS0125	XL CLEANER	4 × 100 mL, Instruction for use

INTENDED USE

XL CLEANER is washing solution contains a surfactant which solubilises proteins and cleans the microcuvettes of ERBA XL analysers intended for *in vitro* diagnostics. For professional use in clinical laboratory only.

PRINCIPLE

The performance of any Erba XL analyser depends on the cleanliness of the microcuvettes. The use of serum or plasma or urine samples results in protein deposition. Over a period of time this may interfere with the assay resulting in erroneous results. XL CLEANER contains a surfactant which solubilises proteins and cleans the microcuvettes.

REAGENT DESCRIPTION AND COMPOSITION

Dodecan-1-ol, ethoxylated	<6.8 %
Sodium dodecyl sulphate	<2.7 %
Sodium hydroxide	<0.1 %

REAGENT PREPARATION

The content of each bottle (100 mL) of XL CLEANER is to be diluted to 10 L with distilled water (100 mL XL CLEANER + 9,900 mL distilled water) before using for cleaning (Cleaning solution Can) in Erba XL analyser.

MATERIAL REQUIRED BUT NOT PROVIDED WITH THE DEVICE

Erba XL analysers:

XL-180	Cat. No. INS00038
XL-200	Cat. No. INS00002
XL-640	Cat. No. INS00008
XL-1000	Cat. No. INS00010

STABILITY AND STORAGE

The reagent is stable till the expiry date mentioned on the label when stored at room temperature 15–25 °C.

WARNING AND PRECAUTIONS

In vitro diagnostic device to be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and / or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: JSJW-PWAS-4J5W-A1NM

Danger

Contains: Dodecan-1-ol, ethoxylated

Hazard statement:

H290 May be corrosive to metals.

H314 Causes severe skin burns and eye damage.

Precautionary statement:

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower.

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Remove contact lenses, if present and easy to do. Continue rinsing.

SAFE DISPOSAL OF DEVICE

Please refer to local legal requirements.

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