

AN 2022_38

ADVISORY NOTICE

Target values for ascorbic acid in URINORM XL control urines

URINORM XL, Catalogue No.: REG00060

Dear customers,

We would like to inform you about possibility to set target values for ascorbic acid in URINORM XL control urines dedicated for quality control of automatic urine analyser LAURA XL combined with UNDEKAPHAN AUTO diagnostic strips.

The presented target values are valid for batches 2111147A (expiration 07/2023) and 2211165 (expiration 03/2024).

Parameter	SCALE 1 + SCALE 2		Units
	URINORM XL N	URINORM XL P	
Ascorbic acid	negative	0,6 – 2,3 / 10 – 40 / 1+ - 3+	mmol/l; mg/dl; ARB

The target values for other parameters presented in URINORM XL instruction for use are unchanging.

In case of any questions, do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.
Technical Product Management
Brno, 22nd of December 2022

