

## ADVISORY NOTICE

### LAURA version 2 software update – version 1.14

### LAURA version 2, Catalogue No.: INS00063

Dear customer,

we would like to inform you that the latest available SW version (1.14) for LAURA version 2 includes two optional scales for microalbumin-creatinine ratio (ACR) reporting and also repair some small bugs in data transmit from device into LIS.

Newly, there are two optional scales of ACR reporting, which can be set by a user. According to Scale 1, three discreet ranges of ACR are reported. For Scale 2, the exact value of ACR between 30-300 mg/g (3.4-33.9 mg/mmol) is reported. The options of ACR reporting are shown in the table below.

ACR REPORTING (SW version 1.14)			
CONV [mg/g]		SI [mg/mmol]	
Scale 1	Scale 2	Scale 1	Scale 2
< 30	< 30	< 3.4	< 3.4
30-300	exact value between 30-300	3.4-33.9	exact value between 3.4-33.9
> 300	> 300	> 33.9	> 33.9

These two options of ACR calculation can be chosen by user: Menu → Setup → Customization → ACR Scale (Scale 1 or Scale 2).

<

setup

Customization



10:57

2022.05.13

colour

clarity

ID Mode

Last result

ACR scale 1

OFF

ON

Erba Lachema s.r.o.



Printed ACR value is calculated from exact values of microalbumin and creatinine results, which are based on calibration curves. There is also a possibility to calculate the ACR from semiquantitative printed values. This option can be allowed only by our technical support.

In case of any questions, please contact your local ERBA team.

Best regards,

Erba Lachema s.r.o.

Technical Product Management

Brno, 24<sup>th</sup> May 2022

**Erba Lachema s.r.o.**

Karásek 2219/1d, 621 00 Brno

DIČ: CZ26918846 -7-

A handwritten signature in blue ink, consisting of a stylized, cursive 'E' followed by a diagonal stroke.



Erba Lachema s.r.o., Karásek 2219/1d, 621 00 Brno, Czech Republic  
Identification number: 269 18 846. Tax identification number: CZ26918846  
Incorporated in the Commercial Register maintained by the Regional Court in Brno, Section C, insert 45458  
Tel.: +420 517 077 111, e-mail: [diagnostics@erbamannheim.com](mailto:diagnostics@erbamannheim.com), [www.eralachema.com](http://www.eralachema.com)