ELite H Clean

Haematology Analyser Cleaner

Cat. No.	Product Name	Packaging (Content)
HEM00023	ELite H Clean	50 mL, Instruction for use

INTENDED USE

ELite H Clean is an *in vitro* diagnostic solution used for cleaning, rinsing and washing of automated haematology analysers ELite 580, H560 and H360. The ELite H Clean should be used by trained professionals in the haematology laboratory.

PRINCIPLE

The product is an alkaline solution that effectively cleans and stabilizes optical and flow parts. Provides clean environment and defends contamination of samples.

DESCRIPTION AND COMPOSITION

Sodium Hypochlorite; Sodium Hydroxide; Surfactant.

PREPARATION

This reagent is ready for use and can be applied straight from the container; no special reagent preparation is necessary. Avoid reagent contaminations.

MATERIAL REQUIRED BUT IS NOT PROVIDED WITH THE DEVICE

Haematology analysers: H360, Cat. No. INS00077
H560, Cat. No. INS00078
ELite 580, Cat. No. INS00071

SPECIFICATION

The product is a clear liquid without particles, precipitate or floccule. Background result: WBC $\leq 0.2 \times 10^9/L$, RBC $\leq 0.2 \times 10^{12}/L$, HGB $\leq 1 g/L$, PLT $\leq 10 \times 10^9/L$.

STABILITY AND STORAGE

The product is stable till the expiry date mentioned on the label when stored at (+2 to +30) °C. Discard opened container after 60 days.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and / or the patient is established.

Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: P3DF-D3C3-QN4R-3N95

Danger

Product contains: Sodium hypochlorite, sodium hydroxide

Hazard statements:

H314 Causes severe skin burns and eye damage.
H400 Very toxic to aquatic life.
H411 Toxic to aquatic life with long lasting effects.

Precaution statements:

P273 Avoid release to the environment.
P280 Wear protective gloves/protective clothing/eye protection.
P301+P330+P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

WASTE MANAGEMENT

Please refer to local legal requirements.

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