Suspensní médium pro NEISSERIAtest

Kat. č.: MLT00025

Pro mikrobiologii

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Dátum revízie: 9. 5. 2019

Suspension medium for NEISSERIAtest

Cat. No.: MLT00025

For microbiology

Suspension medium for NEISSERIAtest is supplementary preparation for MIKROLATEST® NEISSERIAtest kit, which is intended for identification of neisseria, medium is intended for preparation of bacterial suspension.

Suspension medium is supplied sterile in glass tubes. The content of one tube (1.8 ml) is intended for preparing the suspension of one strain.

The preparation of the suspension of a bacterial strain should be carried out according to the procedure recommended for NEISSERIAtest.

1 package of Suspensiom medium for NEISSERIAtest contains 18 tubes with suspension medium for 18 tests.

2 packages (36 glass tubes) serve for 1 package of NEISSERIAtest.

Storage, expiration:

The medium should be stored at the temperature of (+2 to +8) °C. The expiration of the kit is indicated at each packing.

Warning:

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Any dangerous reagents are not included in kits.

Waste disposal:

Contingent rest of the suspension medium should be liquidated in accordance with any other local and national regulations relating to the safe handling of such materials.

Paper and other packaging is necessary to be disposed according to the type of material as sorted waste (paper, glass, plastic).

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