VPtest

VPtest

VPtest

Kat. č.: MLT00041

Pro mikrobiologii

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Cat. No.: MLT00041

For mikrobiology

VPtest strips are recommended for rapid detection of acetoin production (Voges–Proskauer reaction) within 2–4 hours. VPtest strips give results comparable to those obtained by conventional methods, with considerable saving in time. VPtest strips can be used individually as confirmatory test or as supplementary test to MIKROLATEST® kits.

**Principle:**  
The substrate for acetoin formation is sodium pyruvate in paper zone of strip. The strip of VPtest is inserted into saline suspension of strain tested. After incubation for time recommended the reagents of VPT I (α-naphtol) and VPT II (potassium hydroxide) is added. The presence of acetoin is indicated by red colour.

A package of the VPtest contains:

- VPtest strips for the detection of VP reaction for 50 determinations
- Working leaflet
- Storage, expiration:**  
The VPtest should be stored in a refrigerator (+2 to +8) °C until the expiry date indicated on the package. Required material:
- Sterile test tubes (100 x 15) mm with 1 ml of saline
- Reagent for ACETOIN test (Cat. No. MLT00016)

Caution:

- For professional use only

Procedure:

- From a pure culture of strain tested prepare suspesion in saline
- The suspension must have a turbidity recommended in table „Turbidity and incubation“
- Insert the VPtest strip into tube with suspension; both zones of the paper strip must be immersed into saline
- Incubate at 37 °C for time recommended
- After incubation add 3 drops of reagent VPT I and VPT II and shake the tube thoroughly
- Incubate for another 30 min at 37 °C to develop colour reaction
- Read the reaction in accordance with the table „Interpretation of VPtest“

Note:

- The use of different volume of saline, turbidity or incubation conditions as mentioned above may cause false VP reaction
- When using VPtest as a supplementary test to MIKROLATEST® kits remaining suspension for inoculation of kit can be used

Disposal of used material:

After use, all the tubes with strips must be autoclaved.

Waste disposal:

Put packaging waste to recycling or in municipal waste. Used strip liquidate as infectious waste.

Suspension turbidity and incubation time		
	McFarland turbidity scale No.	Incubation (hours)
enterobacteria	1	4
streptococci and enterococci	2	2
staphylococci	2	1.5

Interpretation of VPtest reaction			
Test	Code	Reacion	
		positive	negative
Acetoin	VPT	red, pink	colourless, weak pink

For those who wish to perform their own quality control tests, control bacterial strains are recommended. These strains are supplied in freeze-dried ampoules.

Preparation of McFarland turbidity scale			
No.	Turbidity (x 108/ml)	BaCl <sub>2</sub> (ml)	H <sub>2</sub> SO <sub>4</sub> (ml)
1	3	0.1	9.9
2	6	0.2	9.8

Basic solutions: H<sub>2</sub>SO<sub>4</sub> 1 %  
BaCl<sub>2</sub> · 2H<sub>2</sub>O (ml) 1 %

Results of control strains		
Strain	CCM	VPT
Staphylococcus aureus	1484	+
Micrococcus sedentarius	2699	-

These strains are distributed by  
**Czech Collection of Microorganisms, Masaryk University, Faculty of Science, Kamenice 5, building A25, 625 00 Brno, CZ**  
Tel: +420 549 491 430, Fax: +420 549 498 289  
http://www.sci.muni.cz/ccm, e-mail: ccm@sci.muni.cz

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