Hb-Vario Cleaner Solutions

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INTENDED USE

This Hb-Vario Cleaner Solutions is intended for use with the Hb-Vario Analyser.

REAGENT COMPOSITION

Cleaner	2,5 mmol/l
Tris pufr (pH 8.00)	<0.06 %
Subtilisin	<2 %
Detergent	

Rinse

Sodium azide	0.05 %
--------------	--------

REAGENT PREPARATION

Cleaner and Rinse are liquid, ready to use.

STABILITY / STORAGE

Hb-Vario Cleaner Solutions are stored between 2 and 25 °C. Products are stable until the expiration date stated on the product label.

WARNING AND PRECAUTIONS

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Cleaner contains 0.06 % subtilisin.

Hazard identification in accordance with Regulation (EC) No 1272/2008

Hb-Vario Cleaner

Warning

Hazard statement:

H319 Causes serious eye irritation.
H412 Harmful to aquatic life with long lasting effects.

Precautionary statement:

P273 Avoid release to the environment.
P280 Wear eye protection.
P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P337 + P313 If eye irritation persists: Get medical advice/attention.
UFI: 8HTA-G3AT-7N4A-3XR2

Hb-Vario Rinse

Reagent of the kit is not classified as dangerous.

FIRST AID

In case of an accidental ingestion, wash up the mouth and drink about 0.5 l of water. On eye contact rinse the eye quickly and thoroughly with the jet of tap of water. Contaminated skin should be washed with warm water and soap. In all serious cases of health damage consult a physician.

WASTE MANAGEMENT

Please refer to local legal requirements.

ASSAY PROCEDURE

Refer to the user manual of analyzer Hb-Vario.

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