XL MULTICAL

EN

CALIBRATOR FOR BIOCHEMISTRY

Cat. No.	Pack Name	Content
XSYS0034	XL MULTICAL 4×3	4 × 3 ml
XSYS0122	XL MULTICAL 10×3	10 × 3 ml

INTENDED USE
XL MULTICAL is a lyophilised calibrator based on human serum with chemical additives and tissue extracts of human and animal origin. The concentrations and activities of the analytes are suitable for calibration of appropriate methods on automated clinical chemistry analysers.

VALUE ASSIGNMENT
The calibrator values were determined using the method mentioned in the enclosed value sheet. Values were obtained via determination performed in different laboratories and assigned by a consensus of the results obtained by these laboratories.

RECONSTITUTION
1. Allow the vial to attain room temperature. Remove the screw cap and rubber stopper from the vial to be used. Avoid loss of lyophilizate.
2. Add exactly 3.0 ml of distilled/deionized water.
3. Replace the stopper and cap for each vial and dissolve the contents completely by occasional gentle swirling within 30 minutes. Avoid the formation of foam.

STORAGE AND HANDLING
Unreconstituted lyophilized calibrator is stable till the expiry date mentioned on the label when stored at 2–8 °C.
Stability of the reconstituted calibrator*:
8 hours at 15–25 °C
2 days at 2–8 °C
4 weeks at (-25) – (-15) °C (when frozen once)

***Exceptions:**
Stability of Total Bilirubin in the reconstituted calibrator (when stored protected from light):
6 hours at 15–25 °C
1 day at 2–8 °C
2 weeks at (-25) – (-15) °C (when frozen once)
Stability of Direct Bilirubin in the reconstituted calibrator (when stored protected from light):
3 hours at 15–25 °C
8 hours at 2–8 °C
2 weeks at (-25) – (-15) °C (when frozen once)

Store calibrator tightly capped and protected from light when not in use.

LIMITATIONS
Erroneous results can occur from reconstitution inaccuracies and technical errors associated with the assay procedure.
Do not use the reconstituted calibrator if there is visible evidence of microbial growth in the vial.

WARNING
For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.
Calibrator contains biological material, which is potentially infectious.
Human source material used in the manufacturing of this product has been tested by FDA – approved methods and found non reactive for hepatitis B surface antigen (HbsAg), antibody to Hepatitis C (HCV) and antibody to HIV1/2.
The test procedures do not guarantee that all infectious agents will be detected. Because no test method can offer complete assurance that infectious agents are absent, the material should be handled with same precautions used for patient specimen.

NOTE
Values for XL analysers are now available on websites:
www.erbalachema.com
• Product Support
→XL-analysers: Calibrator/control values for uploading

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