

AN 2022\_32

## ADVISORY NOTICE

**LAURA *version 2* software update – version 1.16**

**LAURA *version 2*, Catalogue No.: INS00063**

Dear customers,

We would like to inform you that the latest available SW version (1.16) for the LAURA *version 2* instrument contains a modified quality control user interface where Bio-Rad Liquichek Urinalysis Control urine has been added. Newly, the QC results for Bio-Rad Liquichek are automatically compared with set target values and the instrument automatically evaluates whether the QC test passed or not passed – the results are reported the same way as for the URINORM control urines.

To switch the quality control to the Bio-Rad Liquichek control urines mode, proceed as follows: MENU → Quality control → Settings → by clicking to MODE user can switch between URINORM or BIORAD control material. After selection of BIORAD mode, the batch and expiration of the used control material can be set.

To enter the target values for Bio-Rad Liquichek, proceed as follows: MENU → Quality Control → Settings → click the MODE to switch to BIORAD mode → click the PAD to display the user interface for entering target values. Available values for negative level are marked "N", pathological level is marked "P". To assign target values for a specific level, mark the desired values - selected values are coloured blue, unselected values are grey. Enter target values for all required parameters. Confirm the setting by pressing ✓.


In case of any questions, do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

Technical Product Management

Brno, 13<sup>th</sup> of December 2022

**Erba Lachema s.r.o.**  
Karásek 2219/1d, 621 00 Brno  
DIČ: CZ26918846 -7-

1/1