

AN 2022\_31

## ADVISORY NOTICE

**LAURA SMART software update – version 3.1 Ma (2.31) and 5.0 Ma**

**LAURA SMART, Catalogue No.: INS00064**

Dear customers,

We would like to inform you that the latest available version of SW – 3.1 Ma (2.31) and 5.0 Ma for the LAURA SMART instrument contains a modified user interface for quality control where Bio-Rad Liquicheck Urinalysis Control urines have been added. Newly, the QC results for Bio-Rad Liquicheck are automatically compared with set target values and the instrument automatically evaluates whether the QC passed or not passed – the results are reported the same way as for the URINORM control urines.

To switch the quality control mode to the Bio-Rad Liquicheck control urines, proceed following steps: MENU → QC TEST → at the bottom of the screen, tap to switch between URINOM or BIORAD modes. After selecting the Bio-Rad control urines, the expiration and batch of the used control material can be set by clicking the SETUP.

Setting of target values for both levels of control material is accessible in ADMIN mode only. To enter target values for Bio-Rad Liquicheck, follow these steps: MENU → QC TEST → click to switch between URINOM or BIORAD modes at the bottom of the screen → SETUP → click the PAD to display the user interface for entering target values. Available values for negative level are marked "N", pathological level is marked "P". To assign target values for a specific level, mark the desired values. Selected values are coloured blue, unselected values are grey. Enter target values for all required parameters. Confirm the settings by pressing OK.

In case of any questions, do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

Technical Product Management

Brno, 07<sup>th</sup> of February 2023

