

ADVISORY NOTICE

36_2023_BICARBONATE Kit

Product name: BICARBONATE, Cat. No. XSYS0100

Dear customer,

We would like to inform you that the following additional text will be stated in the Instruction for use of the kit since the next production lot.

SPECIMEN COLLECTION AND HANDLING

The preferred specimen is from venous blood collected anaerobically in the usual manner for bicarbonate analysis. Bicarbonate content in uncapped tubes decreases approximately 4 mmol/L after one hour. It has been reported that alkalinized serum stored in open cups is stable for up to 4 hours.

In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.
Technical product management

Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Brno 8nd October 2023

Ref.No.AN 2023_36