

ADVISORY NOTICE

42_2023_SW bug in Laura XL SW v.1.2.3.31

Dear customer,

We hereby inform you, that a software bug was discovered in the newest version of Software 1.2.3 for instrument Laura XL. This bug manifests itself as impossibility to perform Cleaning procedure.

To avoid this, we advise you to untick the box in front of the automatic cleaning at fix time by a user with ADMIN rights.

This bug will be fixed in the upcoming Software version. All details are available in the updated software version Technical Note in known bugs chapter - TN-2023-013_3

Applied for:

INSTRUMENT:

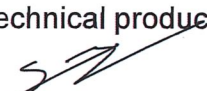
Laura XL

cat.no. INS00065

In case you have any additional question, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.
Technical product management


Brno 23rd 01 2024

Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Ref.No. AN_42_2023