

ADVISORY NOTICE

01_2024_EC90 SW version 1.22.1

Dear customer,

We hereby inform you, that the device software of EC90 is mandatory for update to SW version 1.22.1 issued 5th February 2024. EC 90 analyser SW update from V1.20.8 to V1.22.1 implements bug fixes and new features based on the additional requirements featuring (among others):

- Bugfix: Auto-Print Usage leads to screen Freeze
- Bugfix: 'CAL FOR READY' initiates 1P calibration after 2P calibration fails
- New RFID Support
- The wetting routine is now included as part of commissioning

The SW version 1.22.1 is already included in devices produced from SN 8686-2-202401 onwards.

For SW version 1.22.1, please contact your local service engineer or distributor to set up new SW version from web side [ERBA Knowledge Base](#).

Applied for:

ELECTROLYTE ANALYSER:

EC 90

cat.no.INS00079

In case you have any additional question, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.
Technical product management



Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Brno, 7th February 2024

Ref.No.AN_01_2024