

ADVISORY NOTICE

08_2024_Cleaning of the tubing EC 90

Dear customer,

We would like to inform you that we recommend cleaning of the tubing before every regular change of the EC cartridge or before shutting down for shipment/long inactivation period. The cleaning should be done with Erba Clean I (Cat. No. EHL00042) or 1% NaOCl solution (in distilled water).

However, cleaning should not be carried out more than once a month.

The details of this cleaning routine can be found in the user manual in the chapter 5.6.3 Cleaning the tubing.

Before EC 90 is used again for measuring, it is necessary to carry out a wetting routine with plasma or serum according to user manual chapter 5.6.2.1 Initiate wetting routine.


Affected products

Catalogue number	Product name	Batch number	Production date
INS00079	EC 90 electrolyte analyser		

In case you have any additional question, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.
Technical product management

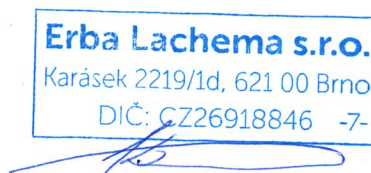

Brno, 25.04.2024

Ref. No.AN_08_2024