

## ADVISORY NOTICE

15\_2024\_ECL 760\_SW\_v2.1e

Dear customer,

We hereby inform you that the device software of ECL 760 has been updated to SW version 2.1e issued on 6<sup>th</sup> May 2024.

ECL 760 device SW update\_v2.1e implements the following changes:

Early clot point and Early clot threshold became adjustable under Item screen upon clicking "Setting" button. Adjustment of these parameters helps to decrease occurrence of EC error flag messages while reporting TT test results for normal samples.

SW version 2.1e is already included in devices produced from SN 0325-71-160223 onwards.

For SW version 2.1e, please contact your local service engineer or distributor to set up the new SW version from the web site [ERBA Knowledge Base](#).

Applied for:

**INSTRUMENT:**

ECL 760

cat.no.INS00070

In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.



**Erba Lachema s.r.o.**

Karásek 2219/1d, 621 00 Brno

DIČ: CZ26918846 -7-

Brno, June 5<sup>th</sup>, 2024

Ref. No.AN\_15\_2024