











ADVISORY NOTICE

13_2024_Plunger replacement



Dear customer,

We hereby request you to remove the plunger from the Accessories box for analyser EC90 due to current version posing a risk of tube and connector damage.

INS00079 EC 90 LIST OF ACCESSORIES

No.	Part Name	Pcs./Set	Picture
1	USB Stick	1	
2	Power Cords: Europe/Russia Power Cord India Power Cord Brazil	1 1 1	
3	Thermal Paper Roll	2	
4	Short Instructions	1	
5	Washing Unit	1	
6	Labeled bottles for washing unit	1	
7	Ring spanner	1	
8	Screwdriver	1	 the image may not correspond to reality
9	Tweezers	1	 the image may not correspond to reality
10	Plunger	1	




If you already own the analyser, a replacement of the plunger will be provided by your distributor free of charge. For analyser up to SN: 8846-2-202401 the replacement will be attached within the analyser transport box.

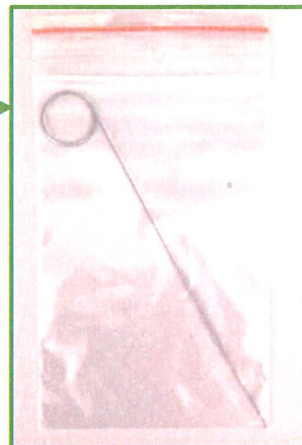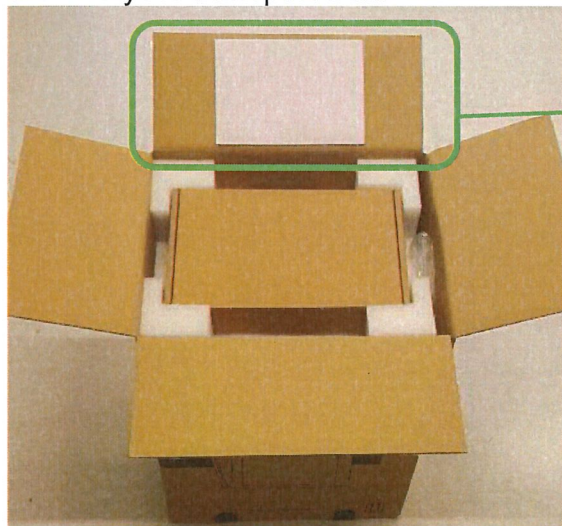

A plunger is included in the accessory box for maintenance and cleaning of the EC 90 instrument. If there is a blockage (e.g. salt crystals) in the fluidic path (tubes; needle; needle block etc...), the plunger shall be used to carefully loosen them. It is important to use it with care so as not to damage any of the above-mentioned components.

The plunger can be used for:


1. Cleaning of the fluidic connector

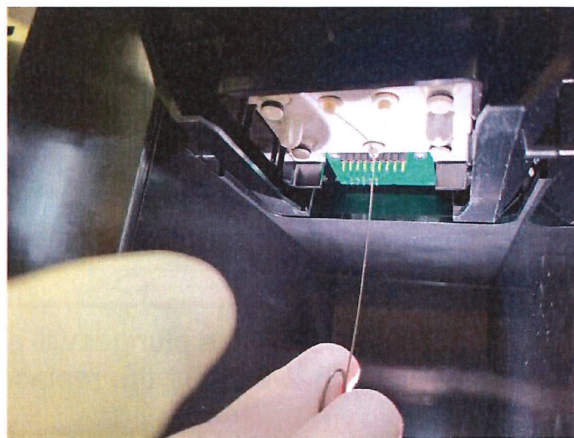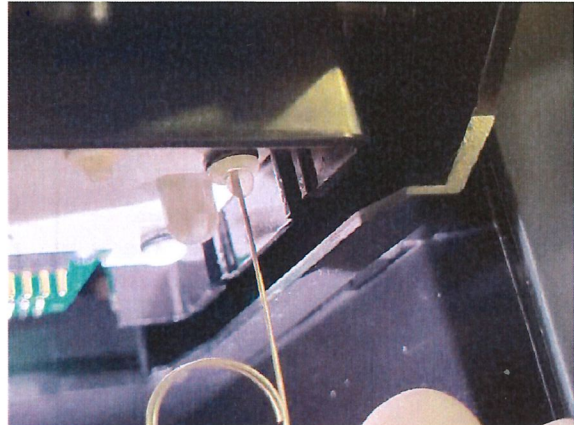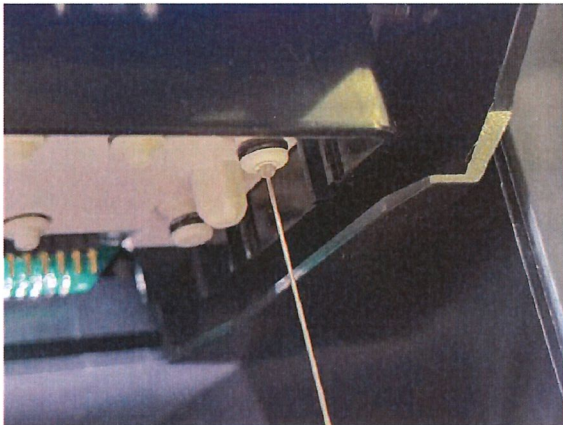

2. Cleaning of the needle, needle guide and needle block using plunger





Cleaning of the needle

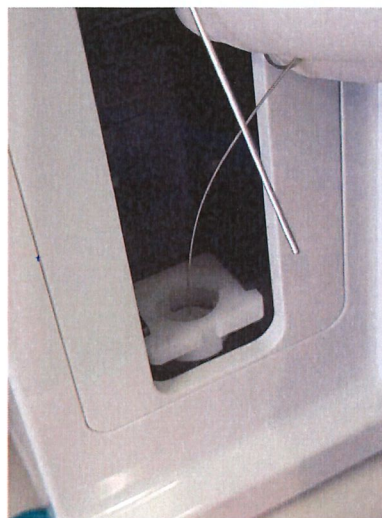


Cleaning of the needle block

3. Cleaning of the air tube

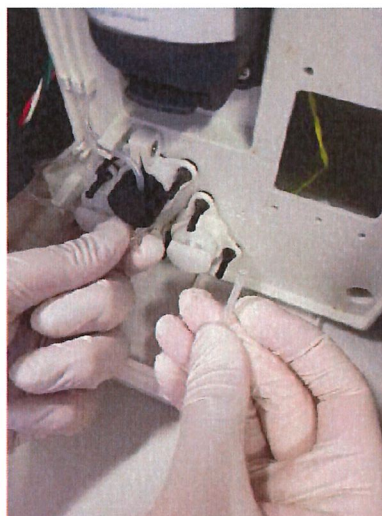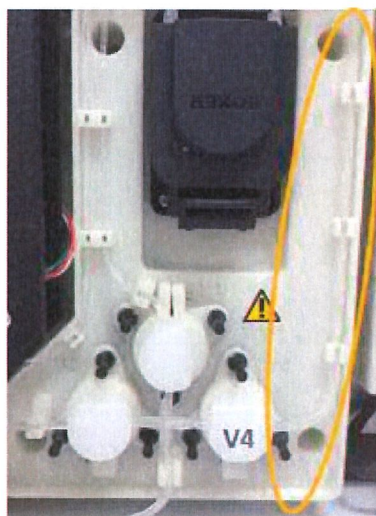

Taking out the air tube

Erba Lachema s.r.o.
IČ: 26918846
DIČ: CZ26918846

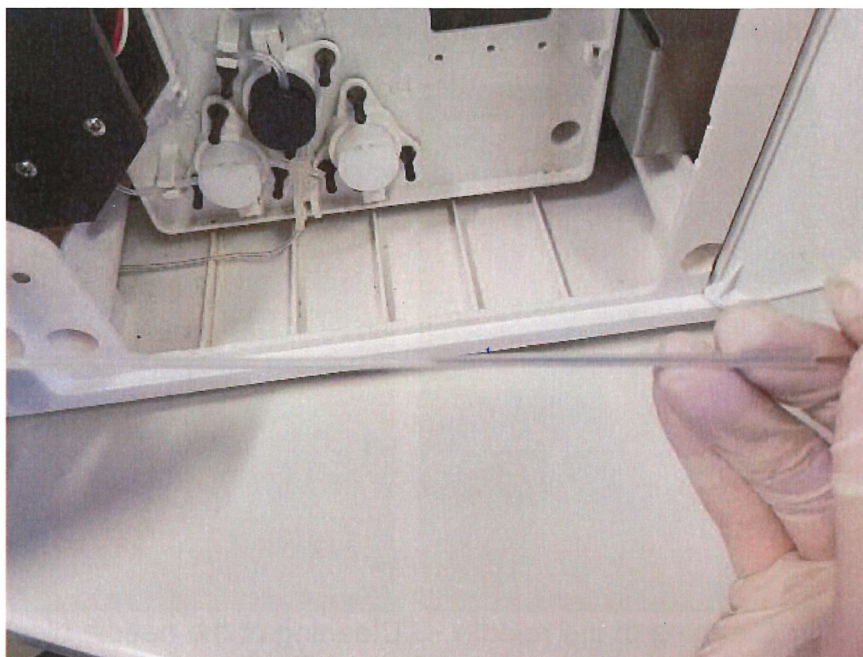

Cleaning of the air tube with plunger (both ends of the tube)

ELECTROLYTE ANALYSER:

EC 90

cat.no.INS00079

In case you have any additional question, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.
Technical product management

Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Brno, June 25th, 2024

Ref. No.AN_13_2024