Erba Hypoclean

EN

Hypochlorite cleaning solution for automated analyzers

IVD

Name	Cat. No.	Package volume
Erba Hypoclean	HEM00016	1 L

For *in vitro* diagnostic use only.

INTENDED USE

Erba Hypoclean hypochlorite cleaning solution is a stabilized and micro-filtered hypochlorite solution for intensive oxidative cleaning, rinsing and washing of hematology analyzers capillaries, tubing and chambers, removing blood component precipitates and lipoprotein deposits on ELite hematology analyzers. Erba Hypoclean hypochlorite cleaning solution should be used only with Erba Lachema reagents. Erroneous results may be obtained if the reagent is applied with third party reagents. Please refer to the instrument User Manual for further information.

INGREDIENTS

Detergents	< 1.0 %
Sodium hypochlorite	< 1.0 %
Sodium hydroxide	< 0.5 %
Stabilizers	< 0.4 %
In ion-free water	

Hazards identification in accordance with Regulation (EC) No 1272/2008

UFI: EQ1A-934U-DN48-VDA6

Danger

Hazard statement:

H290 May be corrosive to metals.

H314 Causes severe skin burns and eye damage.

H412 Harmful to aquatic life with long lasting effects.

Precautionary statement:

P260 Do not breathe vapours.

P273 Avoid release to the environment.

P280 Wear face protection.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

HANDLING PRECAUTIONS

- Avoid contact with eyes, skin and clothing.
- In case of eye contact flush eyes with plenty of water for several minutes and seek medical advice.
- In case of skin contact wash the skin area with water.
- If swallowed do NOT induce vomiting. Drink plenty of water and seek medical advice. Beware of aspiration if vomiting does occur.

USED SYMBOLS

Catalogue Number

Manufacturer

See Instruction for Use

Lot Number

Storage Temperature

Expiry Date

In vitro diagnostic medical device

Content

- Keep the reagent container closed when not in use.
- Avoid any dust or microbial contamination of the reagents.
- Wear laboratory gloves when handling the reagent.
- All body fluid samples should be considered potentially infectious materials. Treat all blood and other potentially infectious materials with appropriate precautions. Use gloves, masks and gowns if blood exposure is anticipated.
- Please refer to the MSDS associated with the reagent.
- Use Good Laboratory Practices (GLP) when handling the reagents.

SPECIMEN COLLECTION AND PREPARATION

Erba Hypoclean reagent is intended for use with blood specimens collected by vein puncture in EDTA anticoagulant. Specimens for hematological analysis may be stored for up to 8 hours at 15–30 °C or up to 24 hours after collection when refrigerated (2–8 °C).

REAGENT PREPARATION

- This reagent is ready to use and can be applied straight from the container, no special reagent preparation is necessary.
- Leave the reagent at room temperature for at least 12 hours.

REAGENT REPLACEMENT

- Person installing the reagent must be a trained laboratory professional.
- Remove the inlet cap from the reagent container to be replaced. Connect the reagent inlet to the new reagent container.
- Be sure that the color on each tube, reagent container label and connector in the back of instrument match.
- Avoid any dust or microbial contamination of the tubing and reagents. Do not pour and mix the remains of a reagent from a container into other one.
- Prime thoroughly the new reagent and measure backgrounds according to the instrument User Manual.
- When installing a new lot of reagent, recalibrate the instrument as specified in your User Manual.

REAGENT STORAGE, STABILITY AND DISPOSAL

Store Erba Hypoclean hypochlorite cleaning reagent between 15–30 °C.

The shelf life of Erba Hypoclean is 24 months from the date of manufacture, if stored at the prescribed temperature range.

Do not use reagent beyond the expiration date printed on the container label.

Discard opened container after 120 days.

Do not use reagent once frozen.

Dispose of waste product, unused product and contaminated packaging in compliance with local regulations.

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