

## ADVISORY NOTICE

27\_2024\_ELAN 30s\_SW\_v2022.11

Dear customer,

We hereby inform you that the ELAN 30s device software has been updated to SW version 2022.011.

ELAN 30s SW\_v2022.11 implements bug fixes and new features:

- In-Well mixing of sample with reagent.  
For non-dilution schedule, sample will be mixed with R1 reagent in well.  
For on-board dilution in tube, (diluted) sample will be mixed with R2 reagent in well.
- If the kit details are not specified for the test, then the device will not allow the batch run to start. The user will be prompted for entire Kit details.
- Soak Time for wash maintained across all strips.
- Validation added for Control & Standards in Test

SW version 2022.11 is already included in devices produced from SN 135042onwards.

For SW version 2022.11, please contact your local service engineer or distributor to set up the new SW version from the web site [ERBA Knowledge Base](#).

Applied for:

**INSTRUMENT:**

ELAN 30s

cat.no: **ELA00001**  
**ELA00002**  
**ELA00003**  
**ELA00004**

In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

**Erba Lachema s.r.o.**  
Karásek 2219/1d, 621 00 Brno  
DIČ: CZ26918846 -7-



Brno, 28<sup>th</sup> August 2024

Ref. No.AN\_27\_2024

