

## ADVISORY NOTICE

39\_2024\_XL-640 Plus\_SW\_v2024.05

Dear customer,

We hereby inform you that the XL-640 Plus device software has been updated to SW version 2024.05 issued on 9<sup>th</sup> September 2024.

XL-640 Plus SW\_v2024.05 includes a new feature and bugs fixes:

1) New features:

- For N! Flag, cutoff change from 0 to -0.01 for absorbance.
- Scheme enhanced with Pairing of R1-R2 Bottles.
- "RUN REPORT" added in Report Menu
- New Reagent Pairing Screen added in Reagent Monitor.

2) Corrective Action:

- Cubic spline test: if the OD of the sample is exactly the same as the OD of the highest calibrator, then the result will be calculated accordingly

3) Lot Calibration Frequency: new SW\_v2024.05 allows user to enter days between 0 and 999.

4) Bugs Fixed:

- Next result after NA (due to "Tech-H/L" flag) was also showing NA in case result is flagged with "H" or "L".
- User is now able to view QC graph on selected in date range more than one day.

SW version 2024.05 is already included in devices produced from SN 645P012 onwards.

For SW version 2024.05, please contact your local service engineer or distributor.  
The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

**INSTRUMENT:** XL-640 Plus cat.no: INS00007, INS000008

In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.  
Brno, 26<sup>th</sup> November 2024

**Erba Lachema s.r.o.**  
Karásek 2219/1d, 621 00 Brno  
DIČ: CZ26918846 -7-



Ref. No.AN\_39\_2024

