

ADVISORY NOTICE

41_2024_ New SW version for LAURA - 1.18

Dear customer,

We hereby inform you that the LAURA urine reader software has been updated to SW version 1.18 issued on 4th December 2024.

This SW update includes two new features:

- The addition of Ukrainian language to enable to switch the user interface to Ukrainian.
- The option to use any QC material which has assigned values for LAURA instrument in addition to Urinorm control urines. To use this feature, proceed as follows: MENU → Quality control → Setup → click on MODE to switch between URINORM and OTHER control material. After selection of OTHER, the batch and expiration of the used control material can be set. Click on PAD to enter the target values of the desired QC material. Available values for negative level are marked "N", pathological level is marked "P". To assign target values for a specific level, select the desired values - marked values are coloured blue, unmarked values are grey. Enter the target values for all required parameters. Confirm the setting by pressing ✓.

The latest version of the SW is available on the [ERBA Knowledge Base](#) website.

Applied for:

INSTRUMENT:


LAURA

Cat. No: INS00063

In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

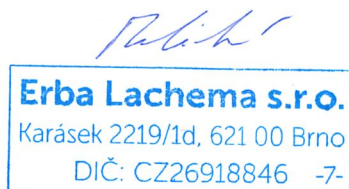

Brno, 4th December 2024

Ref. No.AN_41_2024