

ADVISORY NOTICE

05_2025_XL-180_SW_v2024.C02

Dear customer,

We hereby inform you that the XL-180 devices software has been updated to SW version 2024.C02 issued on 25th November 2024.

Multi XL application software XL180_V2024.C02 is released to include new feature and to address issues reported in previous software version XL180_V2023.C02A.

List of Changes in XL-180 Application software ver. 2024.C02:

1) New features:

- To capture fast reaction (Hook Effect) from the reaction curve of high-value Samples & issue FR Flag.
Check is NOT for Open Tests.
**Pre-requisites for fast reaction check: Load Default ERBA tests & reaction check activation ENC, using parameter management tool, installed from the software CD.
- For N! Flag Cut-off change from 0 to -0.01 for absorbance.
- For N! Flag (Absorbance), cutoff changed from 0 to -0.01
- Set Rerun with decrease volume for 'ABS!' Flag (ABS greater than 3.0)
- Report optimization for printing.

2) CAPA: For test with Cubic Spline curve. When sample OD is Exact same as the highest calibrator OD then result should be calculated appropriately (Not 0)

3) Preventive Measure for Auto Copy Calibration:

- a) Selective/ Repeat/ Copy Calibration buttons are disabled in Calibration screen for copied test with Auto Copy Calibration having at least one standard.
- b) Auto Copy Calibration is not working for standards, if scheduled with only one replicate.

4) Bug Fixed: Even If current calibration is available for the test, CAL* flag is getting attached to the results after doing DCF calibration.

For SW version 2024.C02, please contact your local service engineer or distributor to set up the new SW version from the web site [ERBA Knowledge Base](#).

Applied for:

INSTRUMENT: XL180 cat.no: INS00037, INS00038

Erba Lachema s.r.o.



In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

Erba Lachema s.r.o.

Karásek 2219/1d, 621 00 Brno

DIČ: CZ26918846 -7-

A handwritten signature in blue ink, appearing to be "V. G.", written over a faint, illegible stamp.

Brno, February 6th 2025

Ref. No.AN_05_2025



Erba Lachema s.r.o., Karásek 2219/1d, 621 00 Brno, Czech Republic
Identification number: 269 18 846. Tax identification number: CZ26918846
Incorporated in the Commercial Register maintained by the Regional Court in Brno, Section C, insert 45458
Tel.: +420 517 077 111, e-mail: diagnostics@erba.com, www.erbalachema.com