

ADVISORY NOTICE

06_2025_XL-200_SW_v2024.C04

Dear customer,

We hereby inform you that the XL-200 devices software has been updated to SW version 2024.C04 issued on 21th November 2024.

Application software ver.2024.C04 is released to include new feature & addressed issues reported in previous ASW ver. 2023.C02.

List of Changes in XL-200 Application software ver.2024.C04:

1) New features:

- Stirrer wash for carry over pair (COP) definition.
- For N! flag cut off change from 0 to -0.01 for absorbance.
- To capture fast reaction from the reaction curve of high value sample's reaction compared with the calibrator's reaction curve, sub. Check activated for default test.
Check is not for Open tests
Pre-Requisition for fast reaction check: Load default Erba tests & reaction check activation ENC, using parameter management tool, installed from software CD.
- ABS! Flag: Auto-Rerun with decrease sample Volume: instead of re-run same volume.

2) Improvement:

- Attached W* flag to the results if reagent absent/ VOD/ Froth observed during reagent wash aspiration.

3) Corrective action:

- For a test with Cubic Spline curve, when sample OD is exact same as the highest calibrator OD then result should be calculated appropriately (Not 0).

4) Bugs fixed:

- Auto copy calibration is not getting applied to copied test.
- Blank flash message gets displayed after completion of auto maintenance activity when Russian language is selected.

For SW version 2024.C04, please contact your local service engineer or distributor to set up the new SW version from the web site [ERBA Knowledge Base](#).

Applied for:

INSTRUMENT: XL200 cat.no: INS00001, INS00002

Erba Lachema s.r.o.



In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

Erba Lachema s.r.o.

Karásek 2219/1d, 621 00 Brno

DIČ: CZ26918846 -7-

A handwritten signature in blue ink, appearing to be "J. H.", written over the company stamp.

Brno, February 6th 2025

Ref. No.AN_06_2025



Erba Lachema s.r.o., Karásek 2219/1d, 621 00 Brno, Czech Republic
Identification number: 269 18 846. Tax identification number: CZ26918846
Incorporated in the Commercial Register maintained by the Regional Court in Brno, Section C, insert 45458
Tel.: +420 517 077 111, e-mail: diagnostics@erba.com, www.eralachema.com