

ADVISORY NOTICE

07_2025_XL-640_SW_v2024.C02

Dear customer,

We hereby inform you that the XL-640 devices software has been updated to SW version 2024.C02 issued on 26th November 2024.

Application software MultiXL ver. 2024.C02 is released to include new features, improvement & to address issue reported in previous software ver. 2023.C03.

List of Changes in XL-640 Application software ver.2024.C02:

- 1) New features:
 - For N! Flag, cutoff change from 0 to -0.01 for absorbance.
 - To capture fast reaction (Hook effect) from the reaction curve of high value samples & issue FR flag.
Check is NOT for Open Tests.
**Pre-Requisition for fast reaction check: Load default Erba tests & reaction check activation ENC, using parameter management tool, installed from the s/w CD.
 - ABS! Flag: Auto-Rerun with decrease sample Volume: instead of re-run same volume.
- 2) Corrective Action: For a Test with Cubic Spline Curve, when Sample OD is exact same as the Highest Calibrator OD then result should be calculated appropriately (not 0).
- 3) On Installation with blank data base, under settings - < Calibration policy screen, by default, "Do Not Start Run" option is set and for closed channel reagents, "Discard paired bottle" is ticked by default.
- 4) Bug fixed: Next result after NA (due to "Tech H/L * flag) was also showing NA in case result is flagged with "H" or "L".

For SW version 2024.C02, please contact your local service engineer or distributor to set up the new SW version from the web site [ERBA Knowledge Base](#).

Applied for:

INSTRUMENT: XL640 cat.no: INS00007, INS00008

Erba Lachema s.r.o.



In case you have any additional questions, please do not hesitate to contact us.

Best regards,

Erba Lachema s.r.o.

Erba Lachema s.r.o.

Karásek 2219/1d, 621 00 Brno

DIČ: CZ26918846 -7

A blue ink handwritten signature, appearing to be "V. Lachema", written over a horizontal line.

Brno, February 6th 2025

Ref. No.AN_07_2025



Erba Lachema s.r.o., Karásek 2219/1d, 621 00 Brno, Czech Republic
Identification number: 269 18 846. Tax identification number: CZ26918846
Incorporated in the Commercial Register maintained by the Regional Court in Brno, Section C, insert 45458
Tel.: +420 517 077 111, e-mail: diagnostics@erba.com, www.erbalachema.com