

ADVISORY NOTICE

13_2025_New software version 2024.CO.07

Dear customer,

We hereby inform you that the CHEM 7 device software has been updated to SW version 2024.CO.07 in 12th December 2024.

- 1. Purpose:** Embedded software ver. 2024.CO.07 for CHEM7 is released to include improvements & address issue reported in previous embedded software ver.2024.CO.03.
- 2. List of changes in Application Software ver. 2024.CO.07:**
 - Implemented USB Base handled Barcode scanner.
 - Barcode Scanner enables and disabling in system setup.
 - Scanning ID with barcode scanner in RUN TEST
 - Scanning ID using barcode scanner in REPORTS
 - Barcode Scanner in CALC ITEMS for scanning ID
- 3. Note:**

The released version is compatible only with Cortex M3 PCB - EL20205R3.1 and ADC PCB compatibility is same as Cortex M4 PCB-EL20205R2. CP2102 drivers to be installed for PC communication.

SW version 2024.CO.07 is already included in devices produced from S2408111, S2408112, S2408116, S2408117, S2408118 for domestic market and for export - **S3696**.

For SW version 2024.CO.07, please contact your local service engineer or distributor. The latest version of the SW is available on the [ERBA Knowledge Base](#) website.


Applied for:

INSTRUMENTS:

52000103	INS00014	CHEM-7 - ECZ
50005172	INS00014	CHEM-7
50008058	INS00014	CHEM-7 WITH INCUBATOR

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.


Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-

Brno, April 10th 2025

Ref. No.AN_13_2025

