

ADVISORY NOTICE

14_2025_New software update V2023.01

Dear customer,

We hereby inform you that the XL180 PLUS device software has been updated to SW version 2023.01 in 3rd December 2024.

- 1. Purpose:** Application software ver.2023.01 released to include new features.
Embedded software ver. XL180_2023.RE.01 released to address issue reported in previous ver. XL180_2021.RE.01
- 2. List of changes in Application Software ver. 2023.01:**
Best Viewed resolution is 1152*864
Screen Resolution should be higher than 1152 in width and 864 in height
Absorbance in case of Monochromatic assay (AP) or Bichromatic assay (Ap-As) is negative between reading time such results will be declared as NA and marked with "N!" flag.Note:
- 3. List of changes in the Embedded Software ver 2023.RE.01**
DV calibration succeeded message is getting displayed even if syringe error occurred and no DV activity is getting performed.

SW version **2023.01** and 2023.RE.01 is already included in devices produced from:

Product	Version details	Traceability Export
XL180+	ASW 2023.01 & ESW XL180.2023.RE.01 (XLCD 2567)	S170004, S170005, S170006, S170009, S170013, S170015, S170016

For SW versions, please contact your local service engineer or distributor.
The latest version of the SW is available on the [ERBA Knowledge Base](#) website.


Applied for:

INSTRUMENTS:

50006793	INS00037	XL-180 (with ISE)
50006792	INS00038	XL-180 (without ISE)

In case you have any additional questions, please do not hesitate to contact us.

Best regards,
Erba Lachema s.r.o.


Erba Lachema s.r.o.
Karásek 2219/1d, 621 00 Brno
DIČ: CZ26918846 -7-Brno, April 10th 2025

Ref. No.AN_14_2025